

N95 Mask is Available Here



SAFETYWARE
NIOSH & CE
 Certified Particulate Respirators



3223

6 protect yourself and your loved ones
Six STEPS TO WEAR THE **N95** MASK

<p>STEP 1</p> <p>Wash your hands thoroughly with soap and water or hand sanitizer before putting on the mask.</p>	<p>STEP 2</p> <p>Select a suitable N95 mask that fits well.</p>	<p>STEP 3</p> <p>Determine which side of the mask is the top & side. Hold the N95 mask with a tipped hand and place it firmly over your nose, mouth & chin.</p>	<p>STEP 4</p> <p>Stretch and position top-band high at the back of head. Stretch and position bottom band under the ears.</p>	<p>STEP 5</p> <p>Make sure the Pre-shaped nose cushion fits nicely on your face and adjust the length of headstrap.</p>	<p>STEP 6</p> <p>Perform a fit test by inhaling and exhaling. During exhalation, check for air leakage around face.</p>
--	--	--	--	--	--

Safetyware
 SAFETYWARE GROUP
 Responsibility & Care

SAFETYWARE (SG) PTE LTD (201208598R)

33 Ubi Avenue 3, #06-54 Vertex, Singapore 408868
 Tel : +65-6744 0882 Fax: +65-6842 0882



WWW.SAFETYWARE.COM.SG | INFO@SAFETYWARE.COM.SG