

**10** If no shock advised or after shock delivered, "Start CPR"

**1** Turn AED on

During chest compressions, the ZOLL AED 3 will provide Real CPR Help® prompting for CPR compression quality

**2** "Check responsiveness"

**3** "Call for help"

**4** "Open airway"

**5** "Check breathing"

**6** "If patient is a child, press Child button"

**7** "Attach pads to patient's bare chest"

**8** "Don't touch patient. Analyzing"

**9** If shock advised, "Press flashing Shock button"



For more information on the ZOLL AED 3, visit us at [zoll.com/aed3](http://zoll.com/aed3).

**ZOLL**®